LANAFORM®

Heating Blanket S2

Blanket

Constant heat

Chaleur constante

Constante warmte

INTRODUCTION

Thank you for choosing the LANAFORM® “S2 Heating Blanket” heating pad.

LANAFORM’s “S2 Heating Blanket” heating pad has undergone the rigorous testing required under current European standards. Furthermore, the double heating pad has individual phased temperature control that lets you warm yourself in winter and in damp weather.

PLEASE READ ALL THE INSTRUCTIONS BEFORE USING YOUR “S2 HEATING BLANKET”, ESPECIALLY THIS BASIC SAFETY INFORMATION:

To prevent the risk of burns, electrocution, fire or injury:


- Only use this device according to the instructions given in this manual.
- This device is not intended for use by people (including children) with reduced physical, sensory or mental capabilities, or persons with lack of experience or knowledge, unless a person responsible for their safety can supervise or instruct them beforehand on how to use the device. Children should be supervised to make sure they do not play with the device.
- Do not use accessories that are not recommended by LANAFORM® or that are not supplied with this device.
- If the power cable is damaged, it must be replaced by a special or similar cable available from the supplier or its after sales department.
- Do not use this device if the power plug is damaged, if the device is not working properly, has been dropped on the floor or if it is damaged or has been dropped into water. Have the device examined and repaired by the supplier or its after-sales department.
- Do not carry this device by its power cable or use the cable as a handle.
- Always unplug the device after use or before cleaning.
- Unplug a device that has fallen into water immediately before retrieving it.
- An electrical device must never remain plugged in without supervision. Unplug it when you are not using it.
- Use your heating pad only with the electrical plug provided.
- Do not fold the power cable and do not use the heating pad when it is creased, rolled up or folded.
- Never use this device in a room where aerosols (sprays) are used or in a room or where oxygen is being administered.
- To disconnect the device, turn the button to the «OFF» position, then remove the plug from the mains.
- Do not use safety pins, do not scratch the heating pad with a sharp object and do not beat it with a blunt object. Do not attach the device with pins or other metal objects. Never let anything drop or insert any object into any openings.
- Do not expose your heating pad to the sun and do not iron it. This will prevent damage to the insulation of the electrical circuit.
- This device is not intended to be used in a hospital environment, but for use in the home only.
- Do not use the heating pad with other heating devices or to heat other devices (hot-water bottle, electric blanket, etc.).
- If you are concerned about your health, consult a doctor before using this appliance.
- If you experience any pain when using this device, stop using it immediately and consult your doctor.
- Never use this device on a body part that is swollen or inflamed or if you have a skin rash.
- The control must remain within reach, do not leave the switch hidden under a pillow or any other object.
- Do not wash the heating pad too often to avoid excessive wear and tear. The heating pad should not be washed in the washing machine more than 5 times during its life.
- Do not use the device when it is damp and do not turn it on to dry it.
- When storing your heating pad for a long period, store it in a dry place. Carefully check its condition before operating it again for the first time. If the power cable appears damaged, do not operate the device and take it back to the supplier or its after-sales department.

- This device should not be used by young children, people who are insensitive to heat or on animals.
- The device should not be used on a folding bed or on a bed, a section of which can be adjusted.
- Examine the device frequently to check for any signs of damage to the product or signs of wear. If you notice signs of wear or damage, stop using this product immediately and take it back to the supplier or the its after-sales department.

USING THE HEATING PAD


Diagram and description:

- 1) OFF button
- 2) Position "1" (1st setting)
- 3) Position "2" (2nd setting)
- 4) Position "3" (3rd setting)
- 5) Display screen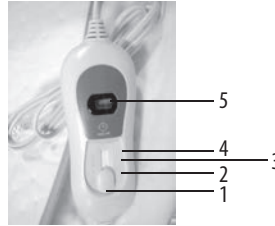

Operating the heating pad:

- Put the pad down flat on the bed. The light side of the heating pad should be against the mattress. The end of the pad without the controller should be placed at the foot of the bed.
- Only put a sheet or fine cotton mattress over the pad so that you can feel the heat.
- Connect each plug to one of the two sockets on each side of the pad and the other end to an electricity supply (AC220-240V). Make sure that there is a power socket nearby and that the power cable is not in a position where the user may trip over it.
- To pre-heat the pad, select the third setting for 15 minutes. The temperature will gradually rise.
- After pre-heating, get on to your heating pad and turn the control button to the heat setting you require. If you use the pad for a long time or if you're going to sleep, select the first setting. It will take about 15 minutes for the pad to reach the required temperature.
- To turn the device off, turn the button to the "Off" position.

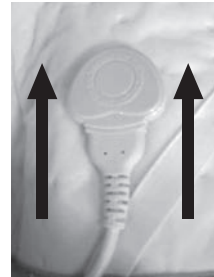Note: The heating pad is set to run continuously for three hours. If you wish to extend that time, it is advisable to let the heating pad cool down first for 5 minutes before using it again.

CLEANING AND MAINTENANCE

When you are not using the heating pad and before cleaning it, follow the instructions below:

- Make sure the control button is in the "Off" position.
- Disconnect the power supply.
- Disconnect the plug linking the control to the device.
- Leave the pad to cool before cleaning.

CLEANING

Small stains can be cleaned with a cloth or a damp sponge and, if necessary, with a little liquid detergent for delicate fabrics.

Warning: the electric heating pad should not be dry-cleaned, bent, dried in a dryer or machine-pressed or ironed.

If the pad is very dirty, it can be washed in a washing machine. Set it to the 30°C programme for delicate fabrics. Use a detergent for delicate fabrics and measure it out according to the manufacturer's instructions.

Do not wash the pad too often to avoid excessive wear and tear. The pad should not be washed in the washing machine more than 5 times during its life.

When drying the pad, do not hang it up with clothes pegs or other such devices.

Do not expose the pad to heat sources such as the sun, a radiator, etc.

Only reconnect the power cable to the pad when the plug and the pad are completely dry.

Never switch the electric pad on to dry it.

STORAGE

If you are not using the pad for a long time, it is advisable to store it in its original packaging in a dry place and not to place any objects on top of it.

ADVICE REGARDING THE DISPOSAL OF WASTE

- All the packaging is composed of materials that pose no hazard for the environment and which can be disposed of at your local sorting centre to be used as secondary raw materials. The cardboard may be disposed of in a paper recycling bin. The packaging film must be taken to your local sorting and recycling centre.
- When you have finished using the device, please dispose of it in an environmentally friendly way and in accordance with the law.

LIMITED WARRANTY

LANAFORM® guarantees this product against any material or manufacturing defect for a period of two years from the date of purchase, except in the circumstances described below.

The LANAFORM® guarantee does not cover damage caused as a result of normal wear to this product. In addition, the guarantee covering this LANAFORM® product does not cover damage caused by abusive or inappropriate or incorrect use, accidents, the use of unauthorized accessories, changes made to the product or any other circumstance, of whatever sort, that is outside LANAFORM®'s control.

LANAFORM® may not be held liable for any type of circumstantial, indirect or specific damage.

All implicit guarantees relating to the suitability of the product are limited to a period of two years from the initial date of purchase as long as a copy of the proof of purchase can be supplied.

On receipt, LANAFORM® will repair or replace your appliance at its discretion and will return it to you. The guarantee is only valid through LANAFORM®'s Service Centre®. Any attempt to maintain this product by a person other than LANAFORM®'s Service Centre will render this guarantee void.

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