Baby Scale BS-500

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User Guide / Bedienungsanleitung

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USER GUIDE

UK The features described in this manual are published with reservation to modifications.

1 Before initial use

1.1 Intended Purpose

The INTEC BS-500 is the ultimate scale for weighing safely your newborn. This easy to use, large LCD display scale is battery-operated scale and makes daily weighing very easy.

1.2 Safety advice

- Always read the safety instructions carefully.
- Follow the basic safety precautions for all electronic equipment when using this Scale.
- Keep this User's Manual for future reference.
- This unit is not waterproof. Don't expose this product to rain or moisture.
- Lay this equipment on a reliable flat surface before setting it up.
- If any of the following situation arises, get the equipment checked by a service technician:
 - The equipment has been exposed to moisture.
 - The equipment has been dropped and damaged.
 - The equipment shows signs of breakage.
 - Do not place the basic unit in a damp room at a distance of less than 1.5 m away from a water source.
- Don't disassemble this scale. The scale doesn't contain any parts that can be repaired by the user.
- Use the scale only for its intended purpose.

1.3 Technical specifications

Operation temperature: 10°C - 40°C

Environmental humidity: <80% R.H.

Max. capacity: 20 kg

Min. capacity: 10 g

Graduation: 10 g

Power supply: 2 x Non - rechargeable AA batteries

Unit change: kg, lb or st

2 Installation

Put batteries in the battery compartment to activate the display:

- Open the battery compartment at the bottom of the unit.
- Insert the 2 AAA non-rechargeable batteries (1,5V) following the polarity as shown.
- Close the battery compartment.

Do not short-circuit and dispose of in fire. Remove the batteries if this device is not going to be used for a long period.

3 Description

1. LCD display
2. ON/OFF/TARA-button
3. UNIT-button

4 Simple operation

- Press the ON/OFF/TARE-button (2).
- Press the **UNIT** - button (3) to select 'Kilogram (**kg**)', 'Stone (**st**)' or 'Pound (**lb**)'.
- Gently place the baby on the pan.
- The babies weight (maximum 20 kg) flashes on the display after a few seconds.

If you use a towel to lay the baby on, it should be placed on the pan before

switching on. If you forgot to do so, press the ON/OFF/TARE-button to eliminate the weight of the towl immediatelly after you have layed down the towl.

If you remove the towl, the scale will display a negative reading.

- Press for a few seconds the ON/OFF/TARE-button (2) to turn the scale OFF or it will turn off automatically in idle mode after 20 seconds.

5 Error messages

Lo – Low power - Replace the batteries

O-Ld – Overload. The weight of the baby is over 20kg.

6 Cleaning

Clean the scale with a slightly damp cloth or with an anti-static cloth. Never use cleaning agents or abrasive solvents.

7 Disposal of the device (environment)

At the end of the product lifecycle, you should not throw this product into the normal household garbage but bring the product to a collection point for the recycling of electrical and electronic equipments. The symbol on the product, user guide and/or box indicate this. Some of the product materials can be re-used if you bring them to a recycling point. By reusing some parts or raw materials from used products you make an important contribution to the protection of the environment. Please contact your local authorities in case you need more information on the collection points in your area.

Batteries must be removed before disposing of the device.

Dispose of the batteries in an environmental manner according to your country regulations.

8 Warranty

8.1 Warranty period

The unit have a 24-month warranty period. The warranty period starts on the day the new unit is purchased. Consumables or defects causing a negligible effect on operation or value of the equipment are not covered.

The warranty has to be proven by presentation of the original purchase receipt, on which the date of purchase and the unit-model are indicated.

8.2 Warranty exclusions

Damage or defects caused by incorrect treatment or operation and damage resulting from use of non-original parts or accessories not recommended by manufacturer are not covered by the warranty.

The warranty does not cover damage caused by outside factors, such as lightning, water and fire, nor any damage caused during transportation. No warranty can be claimed if the serial number on the units has been changed, removed or rendered illegible.

This product is in compliance with the essential requirements and other relevant provisions of the directive 2004/108/EC.

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